VERMEIREN

9060

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

INSTRUCTION MANUAL

Toilet chair Model 9060

TECHNICAL DETAILS

Total height:	82 cm
Height of seat	49 cm
Width:	55 cm
Width of seat:	44 cm
Height of backrest:	35 cm
Depth:	45 cm
Weight:	ca. 7,2 kg
Max. load:	120 kg

CONTENTS OF CONSIGNMENT

Toilet chair + cushioned seat
cushioned lid
- Toilet bucket with cover
Instruction manual

APPLICABILITY

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.

ASSEMBLY INSTRUCTIONS

The toilet chair is delivered fully assembled. Place the chair on level surfaces only and make sure that all four legs stand solidly on the floor. The bucket can now be pushed onto the guide rails below the seat from the left. When not in use the seat can be covered by the cushioned lid supplied.

- ➔ Remove the bucket before transporting the chair.
- ➔ Remove and empty the bucket every time after use.

HINT: Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

SAFETY

- ➔ When setting up the chair and when it is transported, carefully guard against bruising and shearing - danger of injury!

- ➔ Do not grasp sideways further than the length of your arm allows, otherwise the chair can tip over.
- ➔ Before using, check whether all four legs stand evenly on the floor.
- ➔ Do not lean too far forwards -- danger of tipping over !
- ➔ Remove the bucket before transporting the chair.
- ➔ Repairs and disinfection should only be done by authorised persons !
- ➔ Empty the bucket every time after use.

CARE

The seat cushions, arm cushions, frame and bucket can be wiped by using domestic cleaning agents and a damp cloth. Do not use a brush or any other type of scourer.

Chrome parts should be wiped periodically with a dry cloth. Use a mild chrome polish on stubborn dirt.

- ➔ Treatment with hot jets and high pressure equipment is not permitted.

SERVICE

The toilet chair must be checked at least every second year by authorised persons/specialist dealers, and at least every time before resuming its use. The following parameters must be checked:

- Check the frame for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?

The service should only be confirmed in the "Service plan" if at least the above aspects have been checked. If your specialist dealer does not undertake the services which you require, kindly contact the manufacturer. We will gladly refer you to authorised specialist dealers in your vicinity.

DISINFECTION

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

- ➔ Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.
- ➔ Wear suitable protective clothing to prevent skin irritations which might be caused by the disinfectant. Also note the product information of the solutions used.
- ➔ The manufacturer does not accept liability for damage and injury caused

by improper implementation of the disinfection.

Use only disinfectants included in the list of the Robert Koch Institute and the verified procedures of disinfection by scrubbing or wiping.

The current situation in regard to the disinfectants included in the RKI list, can be obtained from the Robert Koch Institute (RKI, home page: www.rki.de).

Toilet buckets and cushioned seats are not meant to be treated hygienically. **Replacement parts can be obtained from your specialist dealer.**

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