VERMEIREN

9063

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

INSTRUCTION MANUAL

Toilet chair Model 9063

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the toiletchair, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your toiletchair. Please remember that observance of our hints would keep your toiletchair in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer. **To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility.**

TECHNICAL DETAILS

Total height:	840 – 985 mm
Height of seat	490 – 635 mm
Width:	620 mm
Width of seat:	460 mm
Height of backrest:	400 mm
Depth:	470 mm
Weight:	ca. 8,5 kg
Max. load:	120 kg

We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg

EXPLANATION OF SYMBOLS

-  Maximum mass
-  Indoor use
-  CE conformity
-  Read the instruction manual before use
-  Observe the safety instructions

CONTENTS OF CONSIGNMENT

Unpack your product and verify if the delivery is complete. Following items should be included:

- Toilet chair + cushioned seat
- Cushioned lid
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

APPLICABILITY

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.


ASSEMBLY INSTRUCTIONS

The toilet chair is delivered fully assembled. Place the chair on level surfaces only and make sure that all four legs stand solidly on the floor. The bucket can now be placed onto the guide rails below the seat. When not in use the seat can be covered by the cushioned lid supplied.

- ➔ Remove the bucket before transporting the chair.
- ➔ Remove and empty the bucket every time after use.

HINT: Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

HEIGHT ADJUSTMENT

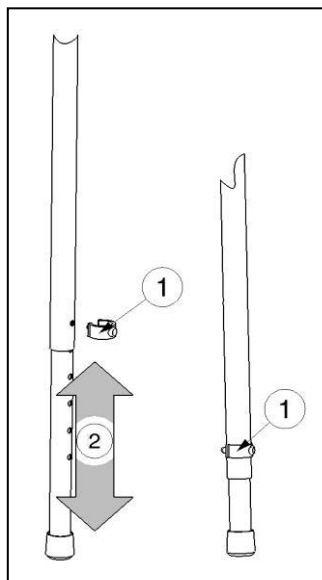1. Remove clip ①.
2. Adjust the leg to the desired position ②.
3. Mount clip ①.

SAFETY

- ➔ When setting up the chair and when it is transported, carefully guard against bruising and shearing - danger of injury!

- ➔ Do not grasp sideways further than the length of your arm allows, otherwise the chair can tip over.
- ➔ Before using, check whether all four legs stand evenly on the floor – risk of tipping over!
- ➔ Do not lean too far forwards -- risk of tipping over!
- ➔ Remove the bucket before transporting the chair.
- ➔ Do not exceed the max load (see technical details)
- ➔ Repairs and disinfection should only be done by authorised persons !
- ➔ Empty the bucket every time after use.
- ➔ Pay attention to the appropriate state of the tips on the legs (damage or lack of them would cause "slip" when sitting down or standing up from a chair)
- ➔ Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- ➔ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

CARE

Cleaning shall be according following instructions:

- **Upholstery / frame**
 - Clean upholstery / frame with a cloth moistened with hot water. Be aware that you do not soak the upholstery / frame.
 - Use a mild commercial detergent for removing stubborn dirt.
 - Stains can be removed by using a sponge or a soft brush.
 - Do not use strong cleaning liquids like solvents, nor use hard brushes.
 - Never clean with steam and/or pressure washers.
 - The manufacturer is not liable for damage caused by improper cleaning agents.

- **Plastic parts**
Clean plastic parts of your toilet chair with commercial plastic cleaners. Only use a soft brush or soft sponge.

Examples of plastic parts are the bucket,

- **Surface layer**
The high quality of the surface layer guarantees optimal protection against corrosion. If the outer coat is damaged by scratching or in some other way,

get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no wetness gets into the tubes.

Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

SERVICE

Lifetime of the toiletchair is influenced by its use, storage, regular maintenance, servicing and cleaning.

The toilet chair must be checked at least every second year by authorised persons/specialist dealers, and at least every time before resuming its use. The following parameters must be checked:

- Check the frame for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?
- Cleaning

The service should only be confirmed in the "Service plan" if at least the above aspects have been checked. If your specialist dealer does not undertake the services which you require, kindly contact the manufacturer. We will gladly refer you to authorised specialist dealers in your vicinity.

DISINFECTION

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

- ➔ Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.
- ➔ Wear suitable protective clothing to prevent skin irritations which might be caused by the disinfectant. Also note the product information of the solutions used.
- ➔ The manufacturer does not accept liability for damage and injury caused by improper implementation of the disinfection.

Use only disinfectants included in the list of the Robert Koch Institute and the verified procedures of disinfection by scrubbing or wiping.

The current situation in regard to the disinfectants included in the RKI list, can

be obtained from the Robert Koch Institute (RKI, home page: www.rki.de).

Toilet buckets and cushioned seats are not meant to be treated hygienically. **Replacement parts can be obtained from your specialist dealer.**

SHIPPING AND STORAGE

The shipping and storage of the toiletchair shall be according following instructions:

- Store in a dry place (between +5°C and +41°C, +41°F and 105.8°F).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the toilet chair from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

WARRANTY

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

DISPOSAL

When disposing of the toiletchair, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials. Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

DECLARATION OF CONFORMITY

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V.

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Commode chairs

Brand: Vermeiren

Type: 9042, 9052, 9060, 9062, 9062 XXL, 9063

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC

and is in conformity with the relevant European harmonized standards:

EN 12182: 1999

SERVICE PLAN

The toilet chair was serviced:

Dealer's stamp

Date :

Dealer's stamp

Date :

Dealer's stamp

Date :

Dealer's stamp

Date :

For service checklists and additional technical information, please see our specialist dealers nearest to you. More information on our website at:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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Vermeiren Suisse S.A.
Vermeiren Nederland B.V.