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### PRODUCT DESCRIPTION:

- manual toilet wheelchair,
- resistant to corrosion,
- Steel chromed frame (colour: grey),
- black soft seat,
- foldable, removable footrest,
- foldable armrests,
- full wheels, rear wheels equipped with brake,
- disassembly allows easy transport.

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**USE:**

Sanitary equipment which enables and facilitates the use of toilet or replaces the toilet. Ideal for short term, indoor patient transportation.

**CONTRAINDICATIONS**

Lack of self-stabilization of the head and trunk.  
Contraindications to a sitting position.

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## ASSEMBLING THE WHEELCHAIR:

Necessary tools : screwdriver, open ended spanner no. 10 and 13.

- Attach the armrests (element no. 1 and 6) – insert them into the wheelchair's frame, turn tight the blocking screws,
- Attach the wheels – insert the wheel bayonet lashing (element no. 10) into the wheelchair frame; attach the wheels equipped with brakes at the back of the wheelchair, the wheels without brakes – in the front, turn tight the blocking screws,
- Attach the footrest (element no. 9) and lean them maximally outside till the moment of latching the blockade. Unfold the footrest platforms (element no. 11). The footrest can be unblocked by pressing the latch and leaning the platforms towards the wheelchair's inside,
- The height of footrest can be regulated by unscrewing the blocking screw placed under the footrest platform. Having chosen an appropriate height, tighten the blocking screw,
- Install the sanitary bin,
- Install the toilet seat and cover it (element no. 4),
- Disassembly – follow all the above-mentioned actions in opposite sequence.

### ATTENTION!!!

Be very careful when folding the wheelchair. The lack of attention can result in pinched fingers between composing parts.

## SAFETY OF USING:

- The wheelchair is designed for indoor use, assistance of a caregiver is necessary to use the wheelchair,
- The wheelchair must be moved by grasping the frame structure. Don't ever move the wheelchair by the armrests or footrests.
- The wheelchair cannot run over architectural barriers (steps, driveways, ramps),

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- Using the wheelchair on surface of grade higher than 5% and not giving some support for user i.e. icy, wet, greasy, highly gritty **IS NOT ALLOWED** – risk of falling.
- The wheelchair is not intended for use as seats in motor vehicles.

#### TECHNICAL INSPECTION

Technical survey needs to be done regularly, at least once in three months. The inspection should be preceded by a careful cleaning of the wheelchair (to reveal any possible damages). During technical survey one needs to:

- Check the condition of brakes,
- Check the safety of screw threads joints,
- Check the state of wheels, judge the wheel's bearing (lift the wheelchair so that the wheel were above the ground, roll the wheel),
- Check the state of bayonet and soldering connections (frame material bursts in soldering surfaces and bayonet connections).

#### ATTENTION!!!

It is unacceptable to use the wheelchair after discovering any visible or detectable damages or when the wear of some parts may cause an accident. All the repairs need to be performed by skilled personnel.

#### EVERYDAY USE AND CONSERVATION

- Always empty the sanitary bin after using,
- Clean the upholstery and frame with a moist cloth,
- Use the toilet detergents according to the instructions given by a manufacturer,
- Remove contaminations like threads, hair off the movable parts of the wheelchair (wheels, footrests).

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Keeping all parts of the wheelchair clean and the way of preserving it, is essential for keeping the wheelchair's original parameters. Keeping the wheelchair in a moist place (steambaths, laundries) or outside together with a lack of hygiene can cause a rapid lost of original parameters of the wheelchair.

#### **USE:**

Every activities such as getting on/off the wheelchair, changing the body position and using the toilet facility **ALWAYS** require:

- Assistance of a caregiver (wheelchair, construction does not allow the user to propel it, to block or unblock the brakes),
- Blocking the wheels.

#### **In order to place the patient in the wheelchair you should:**

- Place the wheelchair in a position which facilitates the patient transport (next to the bed/armchair),
- Block the brakes, fold or remove the footrest,
- Depending on the method of transporting the patient, lift or remove the arrest (it is advised to leave the second armrest to give a patient additional support),
- To transport the patient leave the PVC seat; without container and seat allows comfortable coming close to the toilet; with the sanitary bin wheelchair replaces the toilet seat, check if there is a bin or put the wheelchair over the hole of the toilet.

#### **SPECIFICATION**

Seat width – 44 cm  
 Overall width – 60 cm  
 Wheelchair height – 89 cm  
 Weight – 12 kg (frame weight – 8 kg)  
 User's max weight – 100 kg  
 Wheels diameter – 13 cm

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fax. +48 42 636 52 21

www.mdh.pl <http://www.viteacare.com>

Materials: steel, polyurethane, polythene, rubber.

Recycling of steel and rubber.

Serial number: on the product

Colour: grey (frame), black (seat).

Durability of use: 5 years from date of manufacture.

**Manufacturer:**

mdh sp. z o.o.

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90-349 Lodz , Poland

tel. 0048 42 674 83 84

fax. +48 042 636 52 21

[www.mdh.pl](http://www.mdh.pl) <http://www.viteacare.com>