EN

Product use:

The CoverOne[®] protective helmet is a complementary piece of protective clothing that reduces the risk of droplet-borne infections. **The product is not a toy. It contains small elements. Not suitable for children under 3 years old.**

Method of assembly:

1. Remove protective film from the visor.
2. Protract elastic tape through the cuttings in the strap and adjust the size of the mask to the size of the head (accordingly with the schematics).
3. Attach the visor shield to the retaining strap with the knobs.
4. Shield position can be changed by unscrewing and screwing the knobs on the sides and moving the shield up and down.
5. Pull the visor shield down to cover your mouth and nose.

Visor maintenance:

Visors front shield can be unscrewed from retaining strap. After use, while pulling off the visor, please pay attention to not touch its internal side with dirty hands. In case of hand contact with external side of the visor, precisely wash your hands.

You can reuse the elements of the visor. It is best to maintain them by using alcohol-based cleaning products e.g. window, worktops cleaners with no granules). The use of sponges, hard rubbers or granular detergents can cause tarnishing the

visor's smooth surface and negatively affect its transparency.

For precise disinfection of the visors components, please read all the instructions of the appropriate medical services and use the means indicated by them.

It is highly recommend to avoid washing mask components in water temperature higher than 50 degrees Celsius due to the possibility of deformation. For removing dirt and strong fats you can use fluids based on highly concentrated isopropyl alkohol.

Components of the set (may vary depending on the variants)

1. Protective shield of the visor
2. Retaining strap with foam
3. Two mounting knobs
4. Elastic tape

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