EN

Product intended use:

The semi-face shield made by CoverOne[®] is a completing element of protective clothing, which diminishes the risk of airborne infection transmission. It protects mucous membrane of mouth, eyes and nose and also prevents small mechanical injuries.

Assembly:

1. Remove protective foil from both sides of the shield and a holding strap.
2. Bend the strap according to the diagram and then fix the shield to it using pins and o-rings (silicone seals attached to the set).

Step 1: connect holes "A" together and fix one side of the shield to them.

Step 2: proceed the same way with the holes on the other side of the holding strap.

3. Reeve rubber strings through the holes in the strap and finish them with end fittings.
4. Remember that the shield should be set so as to cover your nose and mouth.

Maintenance:

Remember! The shield must be clean, try not to touch it. While taking it off pay special attention not to touch its inner side. It is especially important as this section of the shield is in direct contact with your respiratory tract. After a contact of your hand with the inner section of the shield disinfect the shield thoroughly.

The shield is a reusable product. The best way to clean it is to use alcohol based products (e.g. disinfection liquids or glass washing liquids). Attention: using of sponges, hard cloth or detergents with granulated substances may result in taking the gloss of the shield and adversely effect its transparency.

For thorough disinfection of the shield read carefully the instruction of the relevant medical services and use the means indicated them. First of all, use the disinfection means based on ethanol (up to 90% of content), without additional substances such as glycerine or solvents. The means including acids or solvents may cause substantial loss of gloss of the shield surface. Due to the risk of deformation scalding of the shield in temperature above 50°C should be avoided.

Set content (it may vary in point 3, depending on the variant)

1. Protective shield
2. Chin holding strap
3. Three fixing pins with o-rings (1 reserve)
4. Two rubber strings
5. Two end fittings

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