



EVERGO™ GIVES PATIENTS THE FREEDOM TO ENJOY LIFE

Introducing the EverGo™ Portable Oxygen Concentrator,
featuring 8-hour battery life, 1050 ml/min oxygen capacity,
and cleared for airline travel.*




OXYGEN DELIVERY THAT YOU AND YOUR PATIENTS CAN RELY ON.

EverGo combines pulse dose breath detection technology and 1050 ml/min of oxygen capacity to make sure your most active patients get the right amount of oxygen. It detects the onset of inspiration and delivers a measured pulse dose of oxygen right when it is needed.

LARGER OXYGEN CAPACITY LETS ACTIVE PATIENTS STAY ACTIVE.


Other portable concentrator units can provide a maximum of only 500–750 ml/min of oxygen. But EverGo delivers 1050 ml/min of oxygen through fixed bolus delivery, increasing the amount of oxygen with each setting. That means your patients have access to more oxygen with each breath – and they have a much lower chance of out-breathing the device.

1050 ml/min

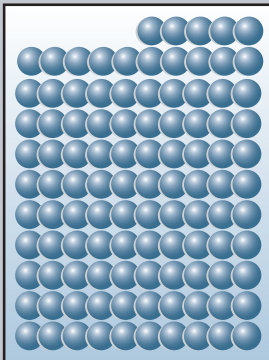

EverGo concentrator

500–750 ml/min

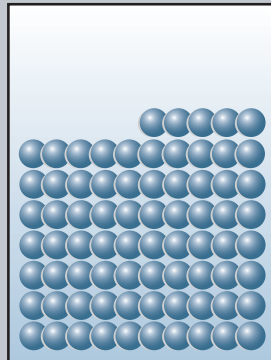

Other portable oxygen concentrators

8-HOUR BATTERY LIFE,* 1050 ML/MIN OXYGEN CAPACITY MEAN MORE FREEDOM FOR PATIENTS.

EverGo is the portable oxygen therapy solution that truly gives your most active patients the freedom they deserve. Its extra-long battery life gives them the freedom to be spontaneous, and its larger oxygen capacity means patients can test their limits – without being limited by the device.

*Based on 20 breaths per minute and a typical device setting of 2.

FEWER TIME CONSTRAINTS, FAA CLEARANCE FOR EVEN MORE FREEDOM.

When patients don't need to wait at home for oxygen deliveries, they're free to explore their world. And EverGo gives them much more world to discover. Weighing less than 4.5 kg, and cleared for air travel by the FAA,* EverGo is ideally suited for patients who are active and love to travel. That leaves your patients free to go wherever they want – whenever they want.

*Note: Airline policies for traveling with an oxygen concentrator vary. Patients need to check with their airlines before flying with EverGo.



SIX DOSAGE SETTINGS LET YOU CUSTOMIZE THERAPY TO YOUR PATIENTS' NEEDS.

SETTING	DOSAGE
1	12 ml
2	23 ml
3	35 ml
4	47 ml
5*	59 ml*
6*	70 ml*

*Based on a respiratory rate of 15 breaths/minute.





Easy-to-use touch screen interface
controls all EverGo functions and clearly shows battery life and prescribed setting

Easy access to batteries
makes them easy to change and charge – plus EverGo can be simultaneously charged and operated

Low maintenance
just one easy-to-remove filter for patients to clean

Non-medical design
looks like an ordinary carry-on bag or camcorder case

Lightweight
weighs less than 4.5 kg

**THE RESIRONICS® FREEDOM SERIES™ –
A NEW FAMILY OF PRODUCTS.
AND A WHOLE NEW WAY TO LOOK
AT OXYGEN THERAPY.**

The EverGo Portable Oxygen Concentrator is part of the Freedom Series of oxygen therapy products - an entire family of products designed to deliver clinically superior treatment and more freedom - freedom from worry, freedom from complicated maintenance, freedom for patients to enjoy life



EVERGO™

ACCESSORIES

EVERGO CARRYING CASE – Available in two colors
BLACK



NAVY BLUE



RECHARGEABLE
LI ION BATTERY
900-102



ACCESSORY CASE
900-250



MOBILE CART
900-100



DC POWER CORD
900-101



AC POWER SUPPLY
900-105



ORDERING INFORMATION

Basic system includes: EverGo Portable Oxygen Concentrator, Carrying Case (choice of color), Two Rechargeable Lithium Ion Batteries, Mobile Cart, DC and AC Power Cords, Accessory Case and User Manual.

CONCENTRATOR SYSTEMS	PART NUMBER
EverGo System with Black Carrying Case, International	900-020
EverGo System with Navy Blue Carrying Case, International	900-022
EverGo System with Black Carrying Case, Canada	900-010
EverGo System with Navy Blue Carrying Case, Canada	900-012
ACCESSORIES	
Gross Particle Filter - 6 Pack	900-150
EverGo Carrying Case - available in two colors	
BLACK	
Canada	900-107
International	900-123
NAVY BLUE	
Canada	900-108
International	900-124
Rechargeable Li Ion Battery	900-102
Accessory Case	900-250
Mobile Cart	900-100
DC Power Cord	900-101
AC Power Supply	900-105
EXTENDED WARRANTY	
One Year Extended Warranty with Compressor	1038653

PRODUCT SPECIFICATIONS

AC Power	Universal Input: 100-240 VAC, 50-60 Hz, 1.0 Amp; Output: 18 VDC, 5 Amp
DC Power	12-18 VDC, 5 Amp
Power Consumption	110 W at 100 VAC (charging), 120 W at 240 VAC (charging); 70 W (not charging)
Oxygen Concentration	89% +/- 3%
Oxygen Delivery Pulse Dose	Settings 1-6 in 0.5 increments
Weight	10 lbs (4.5 kg) with 2 batteries, 8.5 lbs (3.9 kg) with 1 battery
Dimensions	Length: 12 in (30.5 cm) Width: 6 in (15.2 cm) Height: 8.5 in (21.6 cm)
Battery Duration	2 batteries = 8 hours (setting of 2, 20 bpm), 1 battery = 4 hours (setting of 2, 20 bpm)
Sound Level	47 dBA typical at a Setting of 2
Filters	Gross particle, inlet filter, micro-felt filter (bacteria)
Oxygen Purity Alarms	Low oxygen at 83%
Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage/Transport Humidity	-4°F to 140°F (-20°C to 60°C) up to 95% RH non-condensing
Operating Humidity	Up to 95% relative humidity
Outlet Pressure	3 psi
Operating Altitude	Up to 8000 ft
Alarms and Indicators	Low battery at 17 minutes life left; battery status on touch screen and on battery; no breath alarm after 2 minutes
Oxygen Capacity	1.05 liters per minute (pulse dosed)

RESPIRONICS®

Respironics Inc: +1-724-387-5271 fax: +1-724-387-5017
 Respironics Europe, Africa, Middle East: +33-1-47-52-30-00
 Respironics Germany: +49-8152-93-06-0
 Respironics France: +33-2-51-89-36-00
 Respironics Italy: +39-03-62-63-43-1
 Respironics UK: +44-800-1300-845
 Respironics Switzerland: +41-62-745-1750
 Respironics Asia Pacific: +852-3194-2280
 Respironics Australia: +61-2-9666-4444
www.respironics.eu
www.respironics.com

Respironics is a registered trademark, and EverGo and Freedom Series are trademarks, of Respironics, Inc. and its affiliates. Covered by one or more patents. Other patents pending.

©2008 Respironics, Inc. and its affiliates.
All rights reserved.

Brody CY 12/18/07 MCI 4101359 PN 1002349