KSP

yack

MADE IN ITALY

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Technology in motion. The evolution continues

The logo for YACK is written in a bold, blue, sans-serif font with a registered trademark symbol (®) to the upper right of the 'k'.

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YACK was created to move up and down any kind of staircase, even on very narrow stairs, steps with protruding edges, and on winding or spiral staircases. Since it requires limited manoeuvring space, it can also be used in small areas. YACK's intricate design and advanced electronic technology make it an extremely refined, reliable and safe product. An understated and elegant design, paired with extreme manoeuvrability, make this a truly cutting-edge product.

Patented Design.

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Configuration YACK N 912.
Wheeled stair climber for wheelchairs

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This configuration which has width-adjustable lower plates anchored to the body of the machine and width- and height-adjustable upper hooks anchored to the body of the steering column, allows the rapid positioning and anchoring of most wheelchairs on the market today.

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Configuration YACK N 912.
Wheeled stair climber for wheelchairs

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The versatile connection plates of the N 912 make it a nearly universal stair climber, allowing to accommodate and safely transport a wide range of wheelchairs. Practical and very simple to use, it is easily transported thanks to the option of disassembling it into two parts.

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Configuration YACK N 911.
Wheeled stair climber with integrated seat

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N 911 configuration is designed to transport persons who are directly seated on-board the machine. It has a small padded seat with foldable armrests and footrest and height-adjustable headrest. The N 911's reduced size makes it easy to manoeuvre in narrow spaces. The option of disassembling it into several parts makes the machine extremely easy to transport.

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All versions of the machine use the same body and the easy-to-apply accessories make transformations simple... So, it would be easy to detach the N 913's sliding arm, replace it with the backrest and position the seat to transform it into an N 911. Similar manoeuvres allow attaching the sliding arm and wheelchair connection plates to turn it into an N 912. This important feature makes this stair climber capable of adapting to the changing needs of its users. It is a true example of "technology in motion" that is constantly "evolving" in an attempt to make it increasingly easier to overcome every barrier.

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Wheeled stair climber with integrated seat

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Adjustable handle that allows the housing in tight spaces.

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Adjustable steering mechanism or "tiller" that makes the device easy to manoeuvre on stairs. The YACK tiller is fitted with an ergonomic handle that makes it even easier to control.

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Automatic stair-end brakes. This mechanical safety system allows YACK to automatically brake at the edge of the step.

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Ergonomic handwheel for tiller lock.

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Width-adjustable telescoping hooking bar with handwheel adjustments.

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Upper hooks with floating system automatically adjustable according to the inclination of the wheelchair seat to ensure maximum stability.

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The lockable swivel wheels allow the device to be moved on flat surfaces.

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Yack wheeled stair climbers are indicated for disabled persons who need to navigate stairs in buildings without suitable equipment; for this reason, they are suitable for overcoming architectural barriers.

The stair climber consists of a metal structure that moves on stairs using a translating strut system and a pair of wheels for moving on flat surfaces. It has a tiller that allows steering and anchoring a wide range of wheelchairs. The control buttons are ergonomically arranged on the tiller. It is driven by an electric motor with electronic controller that constantly monitors the functioning of the machine, ensuring optimum management of its performance. It is powered by a rechargeable battery that can be extracted and, thus, replaced; its remaining charge is displayed on the control panel. The machine is easy to transport because unhooking the tiller reduces its overall dimensions and weight, making it easy to manoeuvre. The machine body's load-bearing frame is made of high-gauge anodized aluminium, while the steering column is made of painted, double structural steel. The covers are painted steel.

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The device's on-board electronics control its functioning and prevent operation if the conditions of use are not optimal for safety, such as if the residual charge is insufficient or the on-board load is higher than the maximum allowed. A well-designed operator panel makes it extremely easy to set one of the 4 available speeds, select between continuous and single-step operating mode and monitor the residual battery charge.

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Equipped with dual operating controls to ensure easier operation for the carrier.

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On/off jack. When the device is not in use for a prolonged period of time the sleep-mode (stand by) function is activated, followed by an audible warning. The rapid removal of the jack acts as an emergency system that brings the moving machine to an immediate halt.

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External electronic battery charger. The battery can be charged either inside or outside the device, allowing it to be powered with a supplementary battery pack.

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YACK is a class I (CE) medical device manufactured in compliance with:
 - The requirements of European Directive 93/42/EEC as acknowledged by Legislative Decree N46 of 24 February 1997 pertaining to Medical Devices.
 - Legislation related to occupational health and safety
 - Legislative Decree 81/08.

Conforms to the requisites of harmonized standards:
 - CEI EN 60601-1-2
 - UNI EN 980
 - UNI EN 1041
 - ISO 7176-23
 - UNI EN 1441

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YACK's operating phases, step by step. Single step or continuous operation.

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The Company reserves the right to introduce all the technical and/or aesthetical modifications deemed necessary to their products, without prior notice.

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