INTRARACK

This stationary rack system can be used for storage of all kinds of goods.

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**INTRARACK. ORGANISING YOUR AVAILABLE SPACE, EASIER THAN EVER BEFORE**

The Intrarack system is a modular system, made of chromed steel. It is composed of posts and mesh trays. Each post is fitted with an adjustable rubber foot.

This simple storage system provides a great solution for the most diverse products used in healthcare. The customer determines the composition and dimensions of the rack system in view of an optimal use of the available space.

The Intrarack system also offers the possibility to store goods with exceptional dimensions. The loading capacity is 150 kg per shelf, equally divided across the shelf. Stationary rack max. 600 kg.

The system is also available in a mobile version, the Intrarack trolley.

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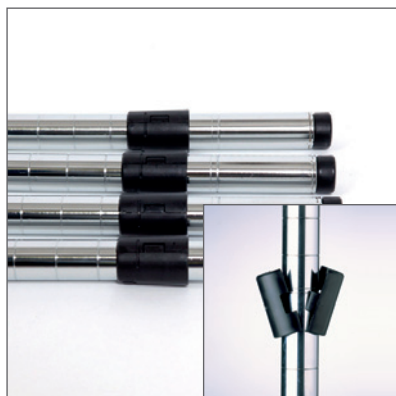
Take the posts and the cone-shaped sleeves out of the box. Join the sleeves at the bottom of the posts. The sleeves lock into the rings of the posts.

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Slide the posts into the corners of the first shelf. The sleeves will secure the shelf.

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Put the posts and the bottom shelf upright. Calculate the height between the shelves. You can do this by counting the number of rings between the shelves.

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Secure the sleeves for the second shelf in the desired position. Slide the second shelf over the posts, onto the sleeves.

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Make sure that each shelf is fitted securely on the sleeves. You may wish to use a small hammer to do this.

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Repeat steps 4 and 5 until you have assembled the entire rack. Adjust the feet if necessary.

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