Multifunktionales Shaver-System 16000 ArthroTec VARIO

Shaver-System der MONOLITH Serie

Multifunctional Shaver System 16000 ArthroTec VARIO

The MONOLITH Line Shaver System

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*The **16000 ArthroTec VARIO** extends the Monolith line by a multifunctional shaver system of the latest generation. The high quality equipment of the shaver and its wide range of functions meet the increased demands and expectations in minimally invasive surgery.*

The combination of high quality materials, noble finish and a glass front panel with an innovative, capacitive touch-screen monitor as well as an easy, intuitive menu navigation emphasises the high technical and aesthetic standards of this shaver system.

The handpieces for the shaver blades and burrs are made of light and resistant titanium, have a powerful motor with a high torque and allow rotation speeds of up to 16000 rpm.

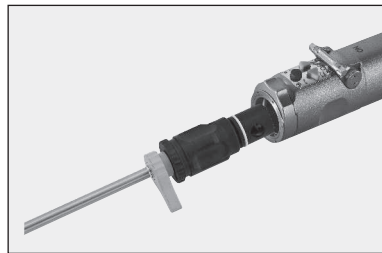
The line is completed by the multifunctional handpiece ARTHRO-CUT vario 13000 pro, which is suitable for a variety of functions used in arthroscopy due to its modular design. Thanks to the easily interchangeable attachments, the drills and saw blades can be operated with the multifunctional handpiece. Attachments for the use of K-wires are also part of the product range.

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The **MONOLITH** Line Shaver System

Further product features

*The **16000 ArthroTec VARIO** control panel has an unrivalled design. By employing latest materials it was possible to integrate the touchscreen monitor entirely into the glass front panel of security glass. The result is a continuous glass front with the unique design of the Monolith line.*

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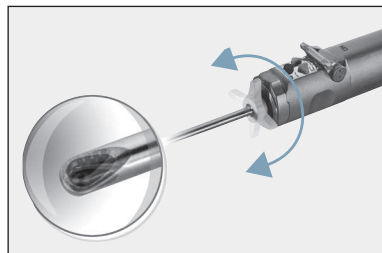
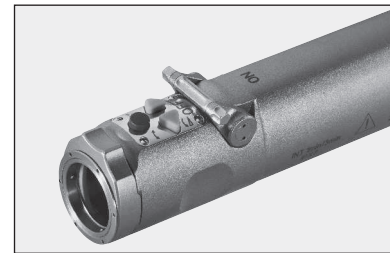
Just-Click Technology

The shaver blade locks in the handpiece easily by gently pushing it into the connector. The automatic handpiece and tool recognition of the system identifies the shaver blade and the presaved performance parameters of the shaver blade are adjusted automatically.

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Because of the use of high-quality titanium the handpiece has a very low weight which together with the oval shaped handle result in very good ergonomic properties, easy handling and working without tiring.

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With the turnable pointer the opening of the blade can be seen extraarticularly at a glance and the cutting window of the blade can be rotated accordingly. After being previously positioned the inner knife stops at the cutting window of the outer sheath when the shaver handpiece is switched off. This allows quick suction or prior suction of tissue without the necessity to reposition manually the inner knife at the cutting window.

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Multifunctional Shaver System 16000 ArthroTec VARIO

The MONOLITH Line Shaver Control Unit

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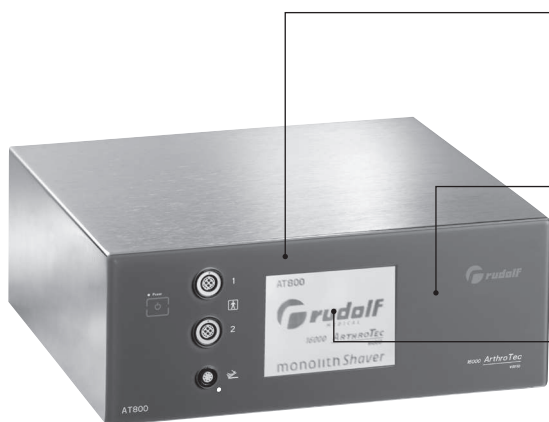
The **16000 ArthroTec VARIO**, designed in the style of the MONOLITH line, combines:

- ▶ Control by capacitive touchscreen. The menu is intuitive due to self-explanatory icons.
- ▶ Saving of up to nine individual user settings. Users can save the preferred torque or the desired rotation speed.
- ▶ Slider function. Simply sweep over the touchscreen to change the rotation speed.

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UNRIVALLED

Security glass front panel with integrated sensor technology

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INDIVIDUAL

Saving up to 9 individual user settings

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Further advantages and product features

The control unit supports the control of the handpieces via the handpiece key pads, the touchscreen of the control unit or the optional foot switch. Thanks to these versatile control options, the user can react optimally to different operation situations.

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Simultaneous connection possibilities for two handpieces and one for the foot-switch allow the easy switching between the handpieces during the operation.

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Accessories for the control unit

Foot switch Ordering no. AT800-310

Waterproof 3-pedal foot switch with pressure-sensitive speed control. The foot switch can be operated by choice in a 2- or 3-pedal mode.

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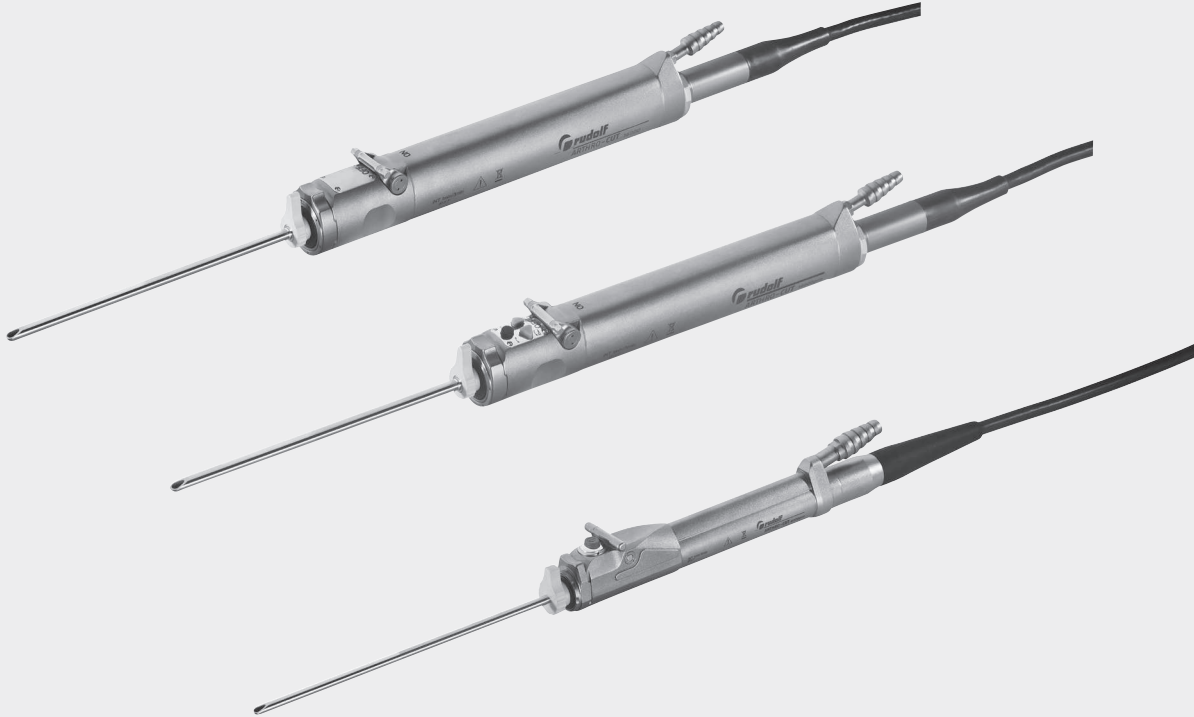
There are **three different handpieces for the shaver system** which were designed taking into account the individual preference of the users. The handpieces impress by their excellent properties in terms of low weight and the high performance of the motor. The housings are made of high quality titanium. The ergonomic handle has been optimised to take into consideration the demands of daily use.

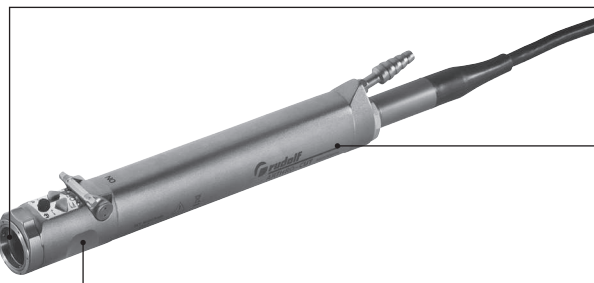
All handpieces are autoclavable at 134°C and are suitable for all commonly used sterilisation procedures like STERRAD® and STERIS®.

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INTELLIGENT

Automatic handpiece and shaver blade detection

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POWERFUL

High torque motor with 22 Ncm and up to 16000 rpm

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ERGONOMIC

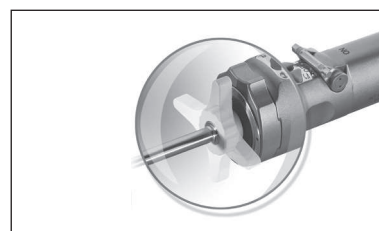
Lightweight handpiece and steplessly variable suction

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Further advantages and product features

The shaver blades have a rotatable pointer at the base of the sheath which indicates and adjusts the blade opening. This allows the user to know the position of the shaver blade opening at any time.

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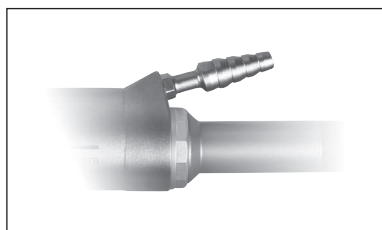
The functions of the shaver blade can be controlled via the key pad on the top of the handpiece. The wide bar for the suction control can be easily operated with the thumb and is suitable for right- and left handed people.

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All handpieces have the just-click technology for automatic handpiece and tool detection. An individually saved rotation speed will be set immediately when the blade is inserted.

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The upwardly angled suction port allows the easy connection of the suction tube.

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Further advantages and product features

ARTHRO-CUT 16000 pro Ordering no.: AT800-321

By using high quality titanium for the housing the handpiece weighs only 380g without cable. The powerful motor achieves a maximum rotation speed of 16000 rpm at 22 Ncm torque. The handpiece ARTHRO-CUT 16000 pro has a key panel for the functions clockwise and counterclockwise rotation, regulation of the motor speed, oscillation and position adjustment of the blade.

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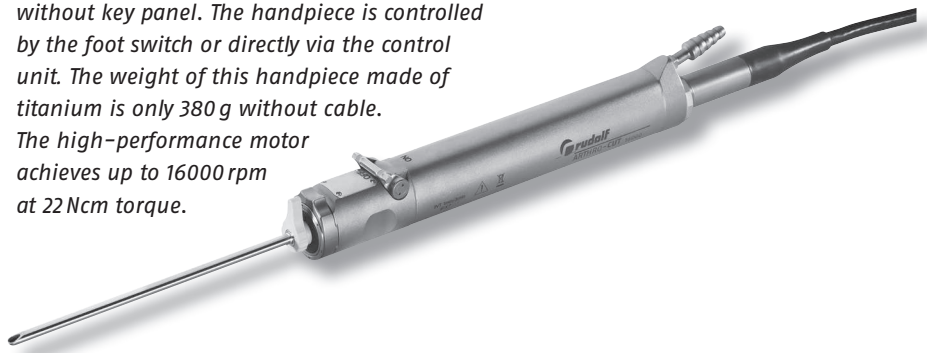
ARTHRO-CUT 16000 Ordering no.: AT800-320

The ARTHRO-CUT 16000 is a handpiece version without key panel. The handpiece is controlled by the foot switch or directly via the control unit. The weight of this handpiece made of titanium is only 380 g without cable. The high-performance motor achieves up to 16000 rpm at 22 Ncm torque.

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
ARTHRO-CUT 6000 Pro Ordering no.: AT800-322

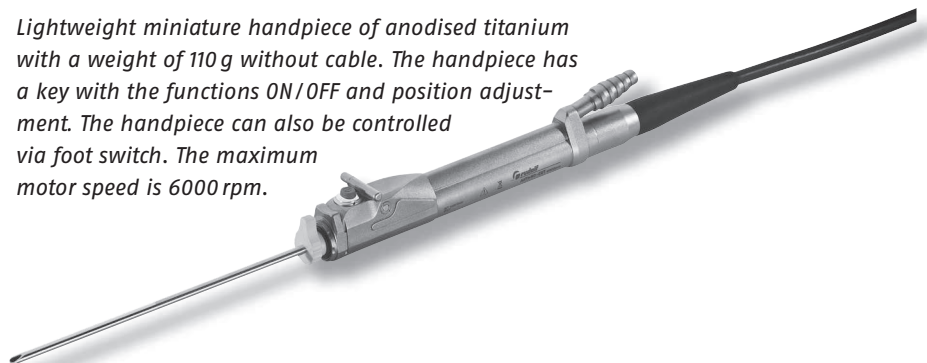Lightweight miniature handpiece of anodised titanium with a weight of 110 g without cable. The handpiece has a key with the functions ON/OFF and position adjustment. The handpiece can also be controlled via foot switch. The maximum motor speed is 6000 rpm.

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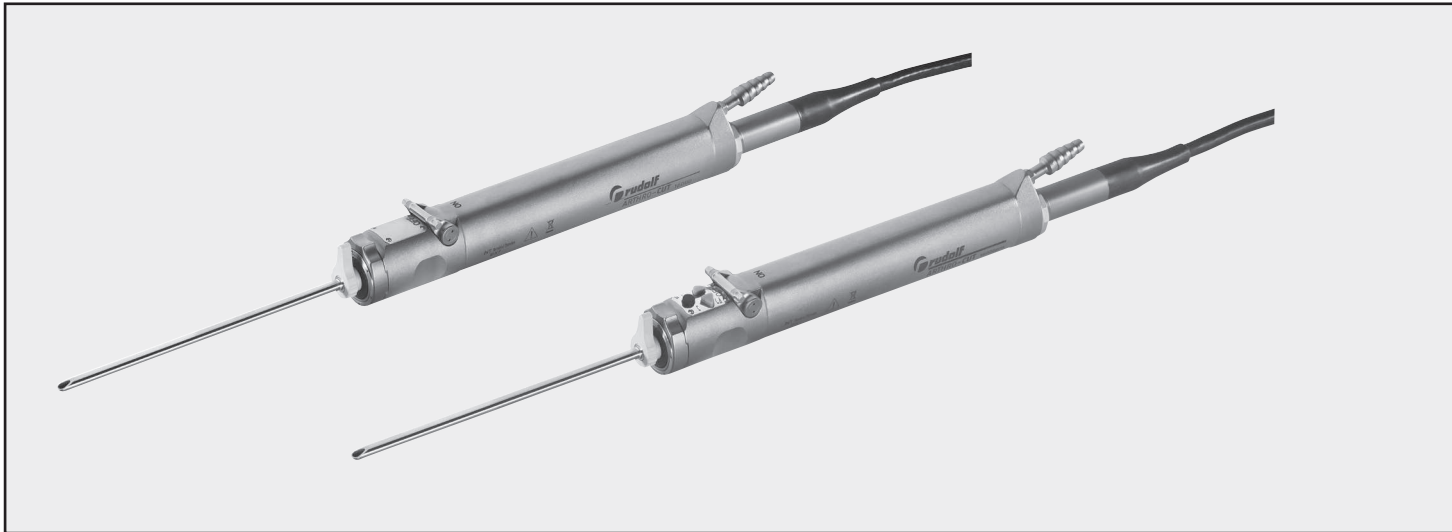
380 g (without cable)
765 g (with cable)

Length

188 mm (handpiece)
250 mm (handpiece incl. rigid cable part)

Degree of protection against the ingress of liquids

IP X7

Suction power regulator

Stepless regulation from 0 – 100%

Number of control keys

ARTHRO-CUT 16000 (AT800-320): 0
ARTHRO-CUT 16000 pro (800-321): 3

Oscillation frequency

4 adjustable levels 0.5/1/1.5/2 Hz

Motor torque

22 Ncm

Cable length

Approx. 3 m

Sterilisation

Autoclavable at 134°C (273°F)
STERRAD®
STERIS®

Mode of operation

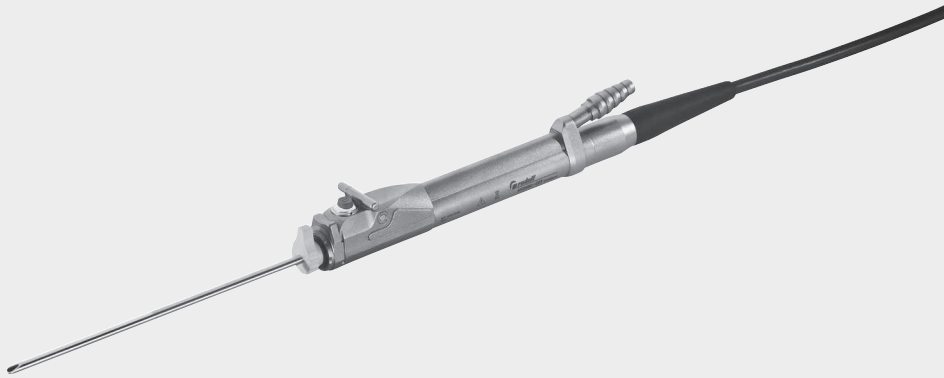
Continuous operation with intermittent loading
INT. 1 min / 3 min

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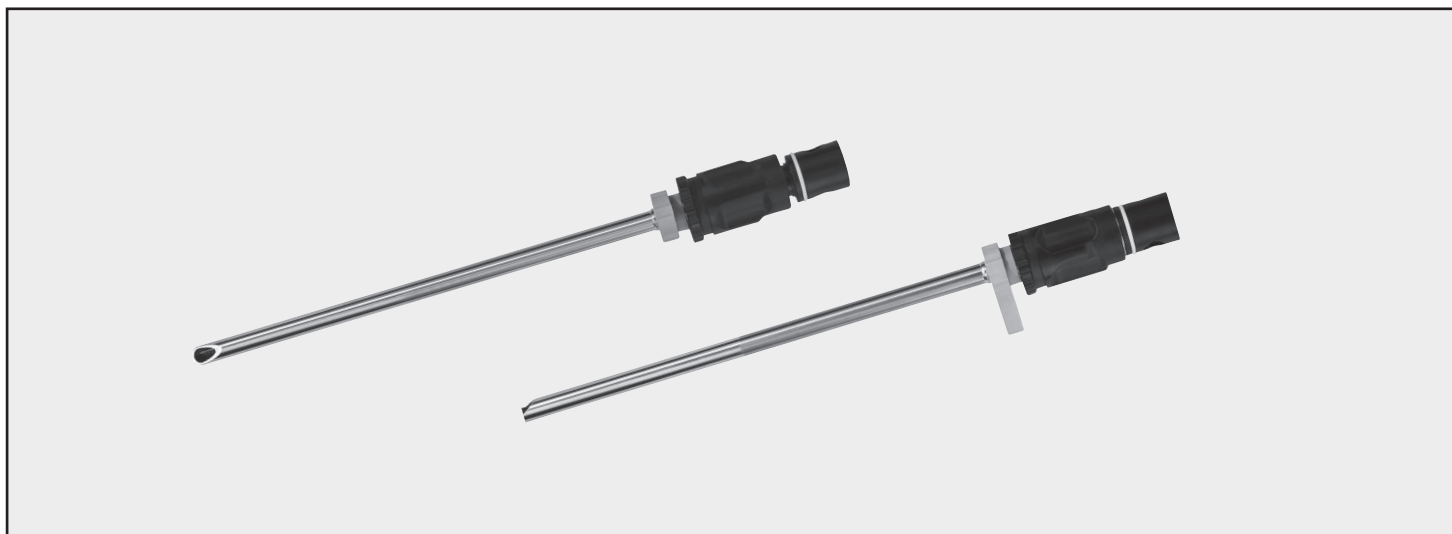
Multifunctional Shaver System Shaver Blades

for the 16000 Arthrocut VARIO Shaver System

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
A good cutting performance and a long service life has been achieved by using high quality hardened stainless steel alloy. All the shaver blades have a pointer on the instrument head which allows to rotate the sheath and check the opening of the blade at a glance. Due to the just-click technology, the handpiece automatically recognises the shaver blade and displays the saved parameters on the control unit.

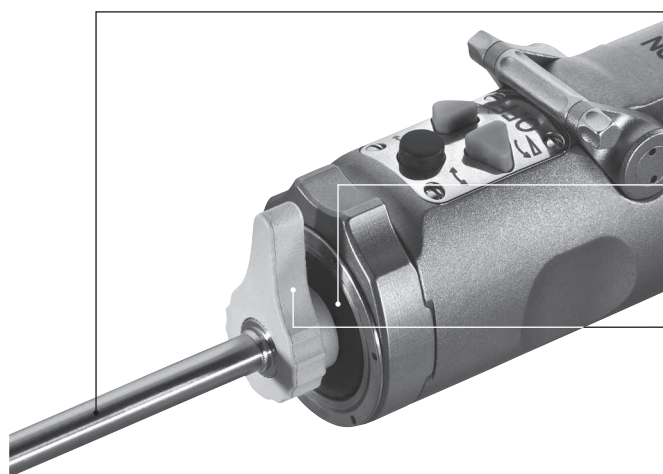All shaver blades have an integrated RFID-chip which transmits operating parameters and data to the respective hand piece.

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INTELLIGENT

**Integrated RFID-chip
for automatic blade
recognition**

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HELPFUL

**Pointer for easy
detection of the
blade opening**

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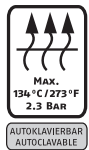
Multifunctional Shaver System Disposable Shaver Blades

for the handpieces ARTHRO-CUT 16000
and ARTHRO-CUT 16000 pro

Beschreibung Description		∅ 3,5 mm	∅ 4,5 mm	∅ 5,5 mm
 Resektor, glatt <i>Resector, smooth</i>		a.N./o.d.	AT801-310*	AT801-311*
 Resektor, gezahnt <i>Resector, toothed</i>		AT801-312*	AT801-313*	AT801-314*
 Resektor, oval <i>Resector, oval</i>		a.N./o.d.	AT801-316*	a.N./o.d.
 Resektor, glatt konvex <i>Resector, smooth convex</i>		a.N./o.d.	AT801-319*	a.N./o.d.
 Resektor, glatt konkav <i>Resector, smooth concave</i>		a.N./o.d.	AT801-322*	a.N./o.d.
 Resektor, aggressive <i>Incisor</i>		AT801-324*	AT801-325*	AT801-326*
 Cutter <i>Cutter</i>		a.N./o.d.	AT801-328*	–
 End Cutter <i>End Cutter</i>		a.N./o.d.	AT801-331*	–
 Kugelfräser mit Schutz <i>Ball burr with guard</i>		–	AT801-332*	–
 Kugelfräser ohne Schutz <i>Ball burr without guard</i>		–	a.N./o.d.	–
 Acromionizer <i>Acromionizer</i>		a.N./o.d.	–	AT801-335* (∅ 8 mm a.N./o.d.)

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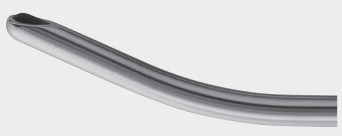
RZ801-999 Cleaning adapter

for the reusable shaver blades of the handpieces
ARTHRO-CUT 16000 and ARTHRO-CUT 16000 pro

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Multifunctional Shaver System ARTHRO-CUT VARIO 13000 pro

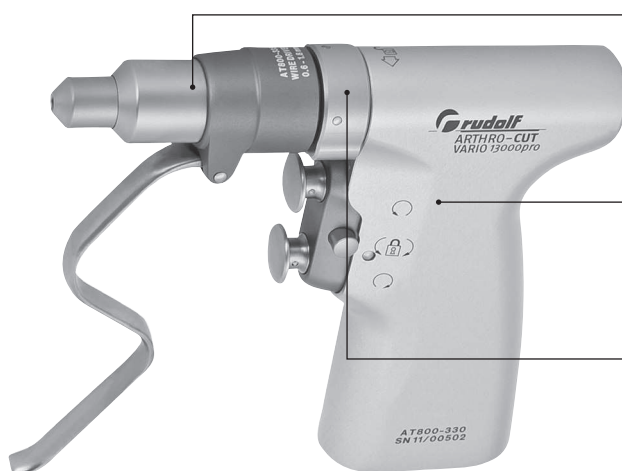
Multifunctional Handpiece

*The **ARTHRO-CUT VARIO 13000 pro** is a powerful multifunctional handpiece for arthroscopy. It is also operated via the 16000 ArthroTec VARIO control unit. Due to two sockets on the control unit the plugged-in handpieces can be used easily by turns, no replugging is therefore necessary. The switching between the handpieces is done automatically either by activating the key pad of the handpiece or the footswitch. For the operation of drills, K-wires and saw blades with the multifunctional handpiece there are available various attachments.*

The multifunctional handpiece ARTHRO-CUT VARIO 13000 pro combines:

- ▶ *Maximum motor speed of 13000 rpm in combination with the sagittal saw attachment*
- ▶ *Powerful, high-torque motor with 30 Ncm torque*
- ▶ *Continuous adjustment of the motor speed*
- ▶ *Lockable keys for right and left rotation*
- ▶ *Removable connection cable*

The multifunctional handpiece with the exchangeable attachments is a valuable completion to the handpieces for shaver blades.

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MULTIFUNCTIONAL

Variable attachment options for the tools

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USER-FRIENDLY

Easy to use **quick locking mechanism** for the attachments

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Multifunctional Shaver System ARTHRO-CUT VARIO 13000 pro

Multifunctional Handpiece

Attachments for ARTHRO-CUT VARIO 13000 pro

Thanks to the various attachments the multifunctional handpiece can be used for drills, saw blades and K-wires. This gives the user the option to use a wide range of tools with only one handpiece.

The attachments can be easily exchanged during the operation due to their smooth and simple locking mechanism.

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Technical Data:

Ordering no.

ARTHRO-CUT VARIO 13000 pro (AT800-3301)
Multifunctional handpiece including connection cable

Max. revolutions per minute

13000 rpm (in combination with the sagittal saw attachment)

Weight (without cable)

495 g

Length (l x w x h)

80 x 25 x 126 mm

Protection degree against the ingress of liquids

IP X7

Number of control keys

2

Motor torque

30 Ncm in drilling mode

Cable length

Approx. 3 m

Sterilisation

Autoclavable at 134°C (273°F)
STERRAD®
STERIS®

Mode of operation

Continuous operation with intermittent loading
INT. 1min / 3min